

Correct sample collection is **IMPORTANT** to ensure timely and valid results. Please read carefully to ensure your training program is clear and effective!

COLLECTION SITES

Post the Visual Aid below in EXAMINATION ROOMS at your Collection Sites.



Note: To order additional copies of the above visual aid (part number L564) contact Gen-Probe at 1-800-523-5001.

Key Points:

- **BLUE SWAB** = Male Urethral and Female Endocervical Specimen Collection
- **WHITE SWAB** = Female cleaning swab only. **NOT FOR COLLECTION!** Discard after use
- Refer to the Specimen Collection Guide for complete collection procedure

BLUE SWAB	ACCEPT
WHITE SWAB	REJECT
2 SWABS	REJECT

Emphasize the APTIMA Unisex Transport Tube must contain only one (1) **BLUE** Swab after collection.